

Check the following	Pass	Fail
Is the unit clean, undamaged, free of excessive wear?	<input type="checkbox"/>	<input type="checkbox"/>
Are there any cracks or loose parts in the housing?	<input type="checkbox"/>	<input type="checkbox"/>
Verify electrodes are connected to the AED Plus and sealed in their package. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>
Are all cables free of cracks, cuts and exposed or broken wires?	<input type="checkbox"/>	<input type="checkbox"/>
Turn the AED Plus on and off and verify the green check indicates ready for use.	<input type="checkbox"/>	<input type="checkbox"/>
Batteries within expiration date. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>
Check for adequate supplies.	<input type="checkbox"/>	<input type="checkbox"/>