

LIFEPAK® 500
Automated External Defibrillator
OPERATOR'S CHECKLIST

Unit Serial No.: _____

Location: _____

This is a suggested checklist for inspecting and checking this device on a daily basis and after each use.

This form may be reproduced.



Instruction	Recommended Corrective Action	Date							
		Initials							

Insert a ✓ in the box after completing each instruction.

1 Examine the AED case, connector, and battery well for:

Foreign substances

Clean the device.

Damage or cracks

Contact authorized service personnel.

2 Examine the battery pins for bending or discoloration.

Discard and replace battery.

--	--	--	--	--	--	--	--	--	--

3 Check expiration date on batteries and therapy electrodes.

Replace if expired.

--	--	--	--	--	--	--	--	--	--

4 Examine the accessory cables for cracked, damaged, broken, or bent connectors or wires.

Replace damaged or broken parts.

--	--	--	--	--	--	--	--	--	--

5 With the battery installed, press On/Off to turn on the AED and look for:

Self-test messages

If absent, contact authorized service personnel.

Momentary illumination of each LED and all LCD segments

If absent, contact authorized service personnel to repair or replace parts.

BATTERY LOW or **REPLACE BATTERY SELF-TEST XX.XX** message

Replace the battery immediately.

Service indicator or **CALL SERVICE** message

Contact authorized service personnel.
