

## Maintaining the AED Plus

Inspect frequently, as necessary. At a minimum monthly inspections are required.

- Check for the green check (✓) in the status indicator window showing that the AED Plus is ready to use.
- Verify that electrodes pads are within their expiration date.
- Verify that batteries are within their expiration date. Typically 5 years from date of installation.
  - Note: there is not an expiration date printed on the batteries. Only a install by date.
- Verify that electrodes are pre-connected to the input connector.
- Verify that supplies are available for use (razor, mask, gloves, extra batteries.)

Document your ZOLL AED Plus inspection on a [AED Inspection tag](#) or separate document

Check the following	Pass	Fail
Is the unit clean, undamaged, free of excessive wear?	<input type="checkbox"/>	<input type="checkbox"/>
Are there any cracks or loose parts in the housing?	<input type="checkbox"/>	<input type="checkbox"/>
Verify electrodes are connected to the AED Plus and sealed in their package. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>
Are all cables free of cracks, cuts and exposed or broken wires?	<input type="checkbox"/>	<input type="checkbox"/>
Turn the AED Plus on and off and verify the green check indicates ready for use.	<input type="checkbox"/>	<input type="checkbox"/>
Batteries within expiration date. Replace if expired.	<input type="checkbox"/>	<input type="checkbox"/>
Check for adequate supplies.	<input type="checkbox"/>	<input type="checkbox"/>

## Cleaning the ZOLL AED Plus

After each use, clean and disinfect the AED Plus with a soft, damp cloth using 90% isopropyl alcohol, or soap and water, or chlorine bleach and water mixture (30 ml/liter water). Do not immerse any part of the AED Plus in water. Do not use ketones (MEK, acetone, etc.) to clean the AED Plus. Avoid using abrasives (e.g., paper towel) on the display window or IrDa port. Do not sterilize the AED Plus.