

## Installing a Battery Pack

The AED Pro unit accepts the following types of battery packs:

- Rechargeable PD 4410-series battery packs
- Disposable sealed lithium manganese dioxide battery packs
- SurePower rechargeable lithium ion battery packs

When the unit displays the message *CHANGE BATTERY*, immediately replace the battery with a fully charged battery pack.

### Before you begin



Be sure the unit is turned off.

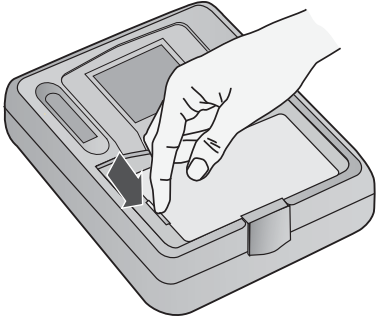

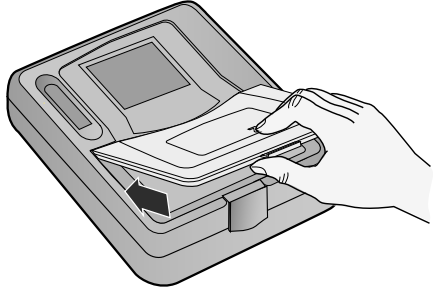
Be sure the battery pack to be installed is fully charged.

**WARNING!** Do not use a rechargeable battery pack if the unit's standby period will exceed 90 days.

### Procedure

To install or replace a battery pack:

Step	Action	Notes
1	Pull the bottom edge of the battery compartment latch outward.  When released, the latch slides upward.	
2	Pull the top edge of the latch toward you and then downward.	
3	Lift the edge of the battery compartment cover.	

Step	Action	Notes
4	Slide out the cover panel.	
5	If a battery is installed, remove it from the compartment by pressing the tab on the battery pack.	
6	Align the tab of the new battery with the finger access area on the left side of the battery compartment, then place the battery into the compartment.	<p>The shape of the battery pack allows it to seat itself properly.</p> 
7	Press the edge of the battery pack until it clicks into place.	
8	Slide the cover panel back into place, and then lower the edge of the cover.	
9	Raise the compartment latch, and tilt the top edge toward the unit.	
10	Press the bottom edge of the latch toward the unit to lock the cover in place.	