


- 1 With the label on the battery facing the AED battery compartment, insert the battery as shown in the drawing.

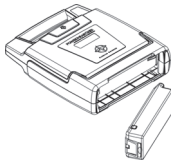

- 2 Push the latched end of the battery firmly into the AED, as shown in the drawing, until the battery snaps into place. The exposed side of the battery should be flush with the outside of the AED case.



- 3 For the Powerheart AED, open the lid for 5 seconds to initiate self-test. If the battery is installed properly, the STATUS INDICATOR will turn GREEN. Close the lid.

OR

Open the lid for 5 seconds to initiate a self-test. If the battery is installed properly, the SMARTGAUGE battery indicator LEDs will illuminate; additionally, the STATUS INDICATOR will turn GREEN. If service is required, then the SERVICE indicator will illuminate.

