

5.6 Operator's Checklist

The following checklist may be used as the basis for an Operator's Checklist. The table should be copied and filled out as recommend by the schedule in the "Routine Maintenance" section. As each item is completed it should be checked off.

Defibtech DDU-100 Operator's Checklist						
Defibtech DDU-100 Serial Number: _____						
Defibtech DDU-100 Location: _____						
Date:						
Check unit and accessories for damage, dirt and contamination. Clean or replace as necessary						
Check that spare battery pack and pads available						
Check that battery pack and pads not past expiration dates						
Check ASI flashing green						
Comments:						
Inspection by: (initials or signature)						

