

HeartSine Samaritan PAD AED Inspection Checklist

<p>Check the Status Indicator If the Status Indicator is not flashing or is flashing red a problem has been detected. Refer to the troubleshooting section of the manual.</p>									
<p>Check the expiration date of the Pad-Pak currently inserted. If the Pad-Pak has exceeded its use by date, remove it from the SAM 300P and replace with a new Pad-Pak. Call or go to AEDUniverse.com at 1-866-517-8243</p>									
<p>Check supplies, accessories and spares for damage or expiration. Replace any accessories found to be damaged or that have exceeded their expiration date.</p>									
<p>Check the exterior of the SAM 300P for cracks or other signs of damage.</p>									
<p>Check that trained responders are aware of the SAM 300P's location and that it is easily accessible for those responders at all times.</p>									
<p>Ensure all trained responders have up to date training for both CPR and AED use. For recommended retraining intervals please consult the organisation or body used to provide the training.</p>									
<p>DATE</p>									
<p>Signature</p>									