

LIFEPAK® 1000 DEFIBRILLATOR

USER'S CHECKLIST



Unit Serial Number _____

Department/Location _____

Instruction	Recommended Corrective Action	Date							
		Initials							
1 Check readiness display for: WRENCH symbol OK symbol Battery level	Contact authorized service personnel. None. Replace if low battery indicated.								
2 Check Use By date on electrode packet.	Replace electrode packet if date has passed.								
3 Check additional supplies.	Replenish as needed.								
4 Check defibrillator for: Damage or cracks Foreign substances	Contact authorized service personnel. Clean the device.								
5 Comments:									